



Technical Data Sheet

Ref. IC-12-B

Type IIR Surgical Mask with Elastic Bands



ICEMEDICS[®]

Technical Information:

Manufacturer Model: LHKL-B-1

Type IIR surgical mask made of three layers of non-woven fabric and filter material. Features a moldable nose clip made of Steel+PP and two soft elastic ear loops for secure fitting.

1st LAYER	2nd LAYER	3rd LAYER
Non-Woven Fabric 25g	Melt-Blown 25g	Non-Woven Fabric 20g

- Nose Clip Length: 11 cm
- Elastic Band Length: 17.5 cm
- Mask Dimensions: 17.5 x 9.5 cm
- Color: Blue
- Latex-Free, Graphene-Free, Fiberglass-Free: Yes
- Non-Irritating: Yes

Standards and Usage:

Compliant with **EU Regulation 2017/745** on **medical devices - Class I**.

Manufactured under the standards of **ISO 13485:2003+AC2007**.

Meets the requirements of **EN 14683:2019+AC:2019**.

Test Number: GOLMOXKC355595L1

Accredited Laboratory: CNAS No. L0412

Bacterial Filtration Efficiency	Breathability/Differential Pressure	Microbial Cleanliness/Bioburden	Splash Resistance Pressure
$\geq 98\%$	$< 60\%$	≤ 30 UF/g	≥ 16

Usage Instructions:

1. Check the mask's condition and expiry date before use.
2. Do not use the mask for more than 4 hours.
3. Wash hands before handling the mask and only touch the straps.
4. Place the mask over the nose and mouth, securing the straps behind the neck.
5. Adjust the nose strip and chin area for a proper fit.
6. Avoid touching or manipulating the mask during use.

Logistics Information:

Labeled in accordance with Standard 14683:2019+AC:2019 and Medical Device Class I.

Presentation:

- Dispenser Pack: 50 units
- Master Carton: 2,000 units (40 dispenser packs)
- Euro Pallet: 25 boxes

EAN Code: 9 6974100420423- Made in China

Basic UDI-DI: 6974100423500AT4

Store in a dry, clean place at temperatures between 5°C and 35°C. Avoid direct sunlight.