



Technical Information:

Manufacturer Model: LHKL-TB-1

Type IIR surgical mask made of three layers of non-woven fabric and filter material. Equipped with a moldable nose strip made of Steel+PP and four tie-on straps for secure fitting.

1st LAYER	2nd LAYER	3rd LAYER
Non-Woven Fabric 25g	Melt-Blown 25g	Non-Woven Fabric 20g

• Nose Clip Length: 11 cm

• Tie-On Strap Length: 41 cm

• Mask Dimensions: 17.5 x 9.5 cm

· Color: Blue

• Latex-Free, Graphene-Free, Fiberglass-Free: Yes

• Non-Irritating: Yes

Standards and Use:

Compliant with EU Regulation 2017/745 on medical devices - Class I.

Manufactured under the standards of ISO 13485:2003+AC2007.

Meets the requirements of EN 14683:2019+AC:2019.

Test Number: GOLMOXKC355595L1 Accredited Laboratory: CNAS No. L0412

Bacterial Filtration Efficiency	Breathability/Differential Pressure	Microbial Cleanliness/Bioburden	Splash Resistance Pressure
>= 98%	< 60%	<= 30 UF/g	>=16

Usage Instructions:

- 1. Check the mask's condition and expiry date before use.
- 2. Do not use the mask for more than 4 hours.
- 3. Wash hands before handling the mask and only touch the straps.
- 4. Place the mask over the nose and mouth, securing the straps behind the neck.
- 5. Adjust the nose strip and chin area for a proper fit.
- 6. Avoid touching or manipulating the mask during use.

Logistics Information:

Labeled in compliance with Standard 14683:2019+AC:2019 and Class I Medical Device.

Presentation:

• Dispenser Pack: 50 units

• Master Carton: 2,000 units (40 dispenser packs)

• Euro Pallet: 25 boxes

EAN Code: 9 6974100420270 - Made in China

Basic UDI-DI: 6974100420270AT4

Store in a dry, clean place at temperatures between 5°C and 35°C. Avoid direct sunlight.